Utasítások

[illegible]

IT

Istruzioni

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106353-2021-12-09_REV0

10

EN		Instructions	
Preparation before cleaning	Instruction	Wipe off the instrument after use to prevent soil and debris to dry onto the instrument. Perform cleaning as soon as possible after use. Do not exceed 2 hours.	
	Cleaning: Manual	Equipment	Soft bristle brushes of various sizes, Ultrasonic bath
		Detergent	Follow agent manufacturers recommendations for concentration and temperature. Manual: Enzymatic or low-alkaline (pH-8) detergent suitable for manual cleaning. Ultrasonic bath: Enzymatic or low-alkaline detergent with minimal foaming characteristics.
		Water quality	Drinking water quality for manual cleaning and distilled or de-mineralized water for ultrasonic bath and final rinse should be used.
Disinfection: Manual	Detergent	Instruction	1. Immerse in freshly prepared cleaning solution as per manufacturer's instructions although maximum 40°C. Clean mechanically with a brush, working beneath the liquid level, until visibly clean. 2. Rinse thoroughly 3. Clean in an ultrasonic bath for a minimum of 5 minutes, using a frequency of 35-45 KHz and power min. 150 W. Make sure the instruments are in an open position. 4. Rinse thoroughly for min 30 seconds. 5. A carefully dry instruments with clean compressed air (Class 1 or better according ISO 8573-1:2010) or lint free wipe. Any remaining water can cause potential rust or corrosion on the instrument surface and into the substrate. 5. Inspect cleanliness. If necessary, repeat from step 1.
Cleaning and disinfection: Automated	Equipment	Washer-disinfecter (validated according to ENISO 15868)	
	Detergent	Enzymatic or low alkaline, suited for medical devices	
	Rinsing Agent	Non-corrosive, neutral cleaning agent with minimal foaming characteristics, suited for medical devices. Follow agent manufacturer's instructions regarding concentration and temperature.	
	Water quality	Drinking water quality for cleaning and deionized or purified water for final rinse/disinfection.	
Inspection and maintenance	Instructions	1. Load the instruments in an open position in the washer-disinfecter. Use a suitable instrument tray. 2. Run program: -Pre-wash in cold water, 2 x 2 min - Main wash with detergent at minimum 55°C, 10 min - Rinse in warm water, 2 x 1 min - Final rinse/disinfection in distilled or de-mineralized water at 90°C, minimum 1 min - Drying at 110°C, minimum 15 min 3. When unloading, control that the instruments are clean. If necessary, repeat from step 1 or use manual cleaning	
Inspection and maintenance		All plastic products have a limited service lifetime and should be replaced at regular intervals. All types of plastic reduce the product's service lifetime. The product should be replaced when its functioning and/or performance no longer satisfies the specifications in this manual.	

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Packaging for sterilization	Equipment Instruction	Standard packaging pouch Pack the instrument in an open position in the pouch. Place pouches flat towards plastic and paper towards paper.
Sterilization	Caution Equipment Instructions	The instrument must be cleaned and disinfected before sterilization. Steam autoclave (validated according to EN 13060, EN 285, EN 17665) Run minimal cycle: Steam temperature Min. 134 °C / 273°F Steam exposure time 4 min Vacuum drying 3 min
Storage		Ensure the instruments are dry before storage. The instrument should be stored in a packed tray, free from dust and moisture at moderate temperatures of 5°C - 40°C.
Transportation		If required, secure transport in closed boxes is recommended in order to avoid any damage to the medical device as well as environmental contamination

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